



SERVICE SOLUTIONS

Cabinet Repair Kits


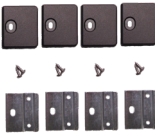


FOR INVACARE PLATINUM AND PERFECTO 2
CONCENTRATORS




Based on feedback from DME provider repair centers regarding stationary concentrator cabinet failures, Oxus investigated an aftermarket solution with an eye toward cost reduction. Oxus has produced a repair kit to allow for cost-effective and professional repair of failed concentrator cabinets while avoiding a complete cabinet replacement.

Included in each kit (see back for installation instructions):

- 4 Structural Brackets
- 4 Cosmetic Top Covers (dark gray)
- 8 Attachment Screws

APPLICATION	DESCRIPTION	OXUS PART #
 Invacare Platinum	 Cabinet Repair Kit	RS-01008
 Invacare PerfectO2	 Cabinet Repair Kit	RS-01009

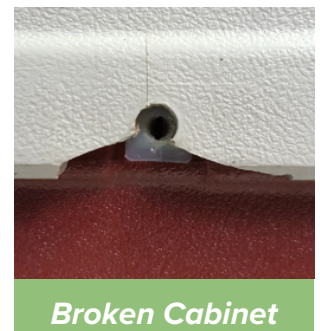


Broken Cabinet

Oxus America
 2046 Brown Rd.
 Auburn Hills, MI 48326
 Phone: 248-475-0925 | Toll Free: 888-475-1568
 sales@oxusamerica.com
 www.oxusamerica.com




Oxus is an engineering, manufacturing, and service company specializing in medical devices and gas separation technology. We are a FDA registered, ISO 13485 Certified OEM Manufacturer and have been in business for over 20 years.

CABINET REPAIR INSTRUCTIONS

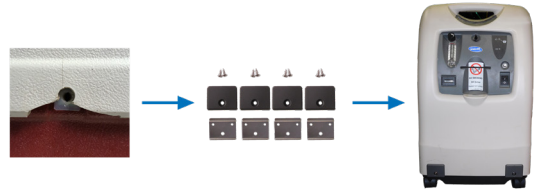

1. Place a *bracket* over the damaged screw mounting hole of the cabinet, centering the bottom hole of the *bracket* over the base insert.

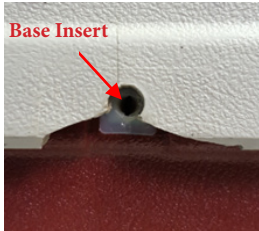

2. Use the original screw to temporarily hold the *bracket* in place.





3. Install the included *self-tapping screws* using a cordless driver.
NOTE: Care should be utilized when using the driver. Make sure to not over-tighten and strip the self-tapped holes.



4. Remove the original screw.



5. Place the cover over the *bracket*, hooking [1] the tabs over the top of the *bracket* and finally rotating [2] the cover towards the bottom of the bracket.

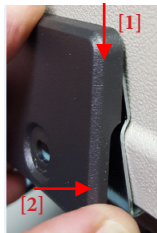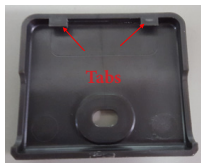

6. Reinstall the original screw.



7. Repeat above steps for the remaining 3 cabinet screw locations.

Oxus America
2046 Brown Rd.
Auburn Hills, MI 48326
Phone: 248-475-0925 | Toll Free: 888-475-1568
sales@oxusamerica.com
www.oxusamerica.com



Oxus is an engineering, manufacturing, and service company specializing in medical devices and gas separation technology. We are a FDA registered, ISO 13485 Certified OEM Manufacturer and have been in business for over 20 years.