





Oxus America
175 Kay Industrial Dr.
Orion Twp., MI 48359
Phone: 248-475-0925
Toll Free: 888-475-1568
sales@oxusamerica.com
www.oxusamerica.com

Instructions for the Use and Return of Stationary Sieve Module Cores - Caire Companion 5L with Integrated Valves and Regulator

Please read the following instructions prior to use:


- Each Oxus sieve is leak and performance tested to ensure purity at or above OEM specifications.
- Minimize the amount of time the sieve is exposed to the environment once the protective cap has been removed.
- Do not cut protective caps from sieves to avoid damaging the fitting.
- Refurbished sieves do not come equipped with the mounting hardware (rubber grommet and plastic bushings) as shown to the right.

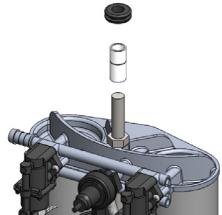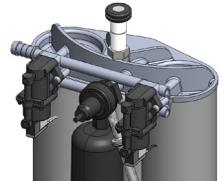

Oxus sieves require return of an acceptable core within 3 weeks of shipment:

- Cores consist of any original OEM parts and/or any out-of-warranty Oxus refurbished parts only. Damaged or unusable cores will not be counted towards core receipt.
- Do not send back mounting hardware on the cores (shown above).
- The following accessories must be returned with the core:

- (1) Pilot valves
- (2) Valve bodies
- (3) Regulator
- (4) Patient tubing with check valve
- (5) Muffler

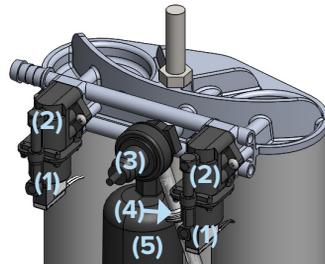

- Sieves with broken or missing components will not be accepted as cores. Please do not return these sieves as cores.
- **Sieve cores MUST be returned in the same packaging as sent by Oxus including all protective spacers or OEM packaging. If cores are not sent back in approved packaging, cores will be damaged and not accepted:**
 - 4 pack box with divider, bottom insert and protective spacers
- Ship core returns to: Oxus America, 175 Kay Industrial Dr. Orion Twp., MI 48359
- See reverse side for Warranty Return Instructions



Oxus America
175 Kay Industrial Dr.
Orion Twp., MI 48359
Phone: 248-475-0925
Toll Free: 888-475-1568
sales@oxusamerica.com
www.oxusamerica.com

Instructions for the Warranty Return of Stationary Sieve Module Cores - Caire Companion 5L with Integrated Valves and Regulator

An RMA (Return Material Authorization) number is required prior to return of the product to Oxus for repair, credit, or replacement.

- *All Oxus stationary sieves come with a 2 year warranty against leaks and rupture.*
- *The Caire Companion 5L replacement sieve module with integrated valves also comes with a 1 year warranty on the valves.*
- *Prior to return of the warranted product, contact Oxus through our designated RMA portal at www.oxusamerica.com to initiate an RMA (must include serial number(s) and description of the problem).*
- *If applicable, Oxus will generate and provide an RMA number based on the ship date. The Oxus RMA number represents acknowledgment for return and tracking of product and does not guarantee warranty replacement.*
- *All RMA sieves **MUST** be capped immediately. **RMA sieves returned without being capped will not be covered under warranty.***
- *RMAs should be shipped in a manner using protective spacers which provides protection. Sieves damaged in shipment due to improper packaging will not be covered under warranty.*
- ***RMA return sieves must be sent separately from return core shipments.***
- *The RMA number must be included on any shipping documentation and on the outside of the shipping container.*
- *Ship RMAs to: Oxus America, 175 Kay Industrial Dr. Orion Twp., MI 48359*



**For more information on our
products and services:**

**Scan the QR Code
OR**

Visit us at www.oxusamerica.com